



MediCare
Colgate



Pessary Range

**Patient focused comfort,
clinician focused outcomes**

t +44 (0)1884 266 666

e info@sterifeed.com

medicarecolgate.com

MediCare Colgate LTD.
Post Cross Business Park,
Kentisbeare, Cullompton, Devon.
EX15 2BB. England

Patient focused comfort, clinician focused outcomes

Our single use, non sterile pessary range consists of over 100 shapes and sizes designed to meet the needs of patients suffering form pelvic organ prolapse & support clinicians to deliver patient focused outcomes.

- ✓ 100% medical grade silicone
- ✓ Latex free
- ✓ Soft & flexible
- ✓ Non-Sterile
- ✓ Single Patient Re-usable




PESSARY RING WITH SUPPORT

Uterine Prolapse grade 1-2 - Cystocele



SIZES (MM)	SIZE CONVERSION	MEDICARE COLGATE CODE (MPC)	NHS PRODUCT CODE (NPC)	UNITS (UOI)
45mm	0	RO24500S	FUI85018	1
51mm	1	RO25101S	FUI85006	1
57mm	2	RO25702S	FUI85022	1
64 mm	3	RO26403S	FUI85042	1
70 mm	4	RO27004S	FUI85004	1
76mm	5	RO27605S	FUI85007	1
83mm	6	RO28306S	FUI85046	1
89mm	7	RO28907S	FUI85057	1
95mm	8	RO29508S	FUI85045	1
100mm	9	RO210209S	FUI85023	1

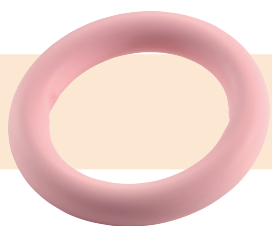

PESSARY RING WITHOUT SUPPORT

Uterine Prolapse grade 1-2 - Cystocele



SIZES (MM)	SIZE CONVERSION	MEDICARE COLGATE CODE (MPC)	NHS PRODUCT CODE (NPC)	UNITS (UOI)
45mm	0	RO14500	FUI85025	1
51mm	1	RO15101	FUI85047	1
57mm	2	RO15702	FUI85024	1
64mm	3	RO16403	FUI85010	1
70mm	4	RO17004	FUI85009	1
76mm	5	RO17605	FUI85026	1
83mm	6	RO18306	FUI85048	1
89mm	7	RO18907	FUI85049	1
95mm	8	RO19508	FUI85012	1
100mm	9	RO110209	FUI85011	1



PESSARY DISH WITH SUPPORT

Uterine Prolapse grade 1-2 / 2-3 - Cystocele - Stress Incontinence



SIZES (MM)	SIZE CONVERSION	MEDICARE COLGATE CODE (MPC)	NHS PRODUCT CODE (NPC)	UNITS (UOI)
55mm	1	NDSH055501S	FUI85021	1
60mm	2	NDSH056002S	FUI85005	1
65mm	3	NDSH056503S	FUI85058	1
70 mm	4	NDSH057004S	FUI85043	1
75 mm	5	NDSH057505S	FUI85044	1
80 mm	6	NDSH058006S	FUI85040	1



PESSARY DISH WITHOUT SUPPORT

Uterine Prolapse grade 1-2 / 2-3 - Cystocele - Stress Incontinence



SIZES (MM)	SIZE CONVERSION	MEDICARE COLGATE CODE (MPC)	NHS PRODUCT CODE (NPC)	UNITS (UOI)
55mm	1	NDSH055501	FUI85019	1
60mm	2	NDSH056002	FUI85020	1
65mm	3	NDSH056503	FUI85041	1
70mm	4	NDSH057004	FUI85008	1
75mm	5	NDSH057505	FUI85052	1
80mm	6	NDSH058006	FUI85017	1




PESSARY GELLHORN WITH SHORT STEM

Uterine Prolapse grade 2-3 - Cystocele - Rectocele



SIZES (MM)	SIZE CONVERSION	MEDICARE COLGATE CODE (MPC)	NHS PRODUCT CODE (NPC)	UNITS (UOI)
38mm	0	NGO43800S	FUI85033	1
45mm	1	NGO44501S	FUI85055	1
51mm	2	NGO45102S	FUI85035	1
57mm	3	NGO45703S	FUI85037	1
64mm	4	NGO46404S	FUI85054	1
70mm	5	NGO47005S	FUI85031	1
76mm	6	NGO47606S	FUI85016	1
83mm	7	NGO48307S	FUI85032	1
89mm	8	NGO48908S	FUI85000	1
95mm	9	NGO49509S	FUI85053	1

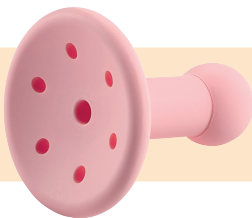

PESSARY GELLHORN WITH LONG STEM

Uterine Prolapse grade 2-3 - Cystocele - Rectocele



SIZES (MM)	SIZE CONVERSION	MEDICARE COLGATE CODE (MPC)	NHS PRODUCT CODE (NPC)	UNITS (UOI)
38mm	0	NGO33800	FUI85030	1
45mm	1	NGO34501	FUI85015	1
51mm	2	NGO35102	FUI85014	1
57mm	3	NGO35703	FUI85051	1
64mm	4	NGO36404	FUI85029	1
70mm	5	NGO37005	FUI85050	1
76mm	6	NGO37606	FUI85013	1
83mm	7	NGO38307	FUI85028	1
89mm	8	NGO38908	FUI85027	1
95mm	9	NGO39509	FUI85059	1



PESSARY CUP WITH SUPPORT

Uterine Prolapse grade 1-2 - Cystocele - Stress Incontinence



SIZES (MM)	SIZE CONVERSION	MEDICARE COLGATE CODE (MPC)	NHS PRODUCT CODE (NPC)	UNITS (UOI)
55mm	1	NCP065501S	FUI85001	1
60mm	2	NCP065502S	FUI85002	1
65mm	3	NCP065503S	FUI85039	1
70mm	4	NCP065504S	FUI85056	1
75mm	5	NCP065505S	FUI85034	1
80mm	6	NCP065506S	FUI85060	1




PESSARY CUP WITHOUT SUPPORT

Uterine Prolapse grade 1-2 - Cystocele - Stress Incontinence



SIZES (MM)	SIZE CONVERSION	MEDICARE COLGATE CODE (MPC)	NHS PRODUCT CODE (NPC)	UNITS (UOI)
55mm	1	NCP065501	FUI85061	1
60mm	2	NCP065502	FUI85062	1
65mm	3	NCP065503	FUI85038	1
70mm	4	NCP065504	FUI85036	1
75mm	5	NCP065505	FUI85003	1
80mm	6	NCP065506	FUI85063	1

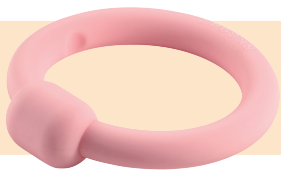


RING WITH KNOB

Uterine Prolapse grade 1-2 / 2-3 - Cystocele - Stress Incontinence



SIZES (MM)	SIZE CONVERSION	MEDICARE COLGATE CODE (MPC)	NHS PRODUCT CODE (NPC)	UNITS (UOI)
45mm	0	PERK00	FUI85081	1
51mm	1	PERK01	FUI85077	1
57mm	2	PERK02	FUI85078	1
64mm	3	PERK03	FUI85102	1
70mm	4	PERK04	FUI85079	1
76mm	5	PERK05	FUI85100	1
83mm	6	PERK06	FUI85076	1
89mm	7	PERK07	FUI85101	1

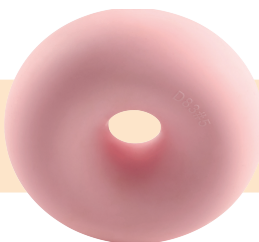

DONUT

Uterine Prolapse grade 2-3 - Cystocele - Stress Incontinence



SIZES (MM)	SIZE CONVERSION	MEDICARE COLGATE CODE (MPC)	NHS PRODUCT CODE (NPC)	UNITS (UOI)
51mm	0	PED00	FUI85098	1
57mm	1	PED01	FUI85096	1
64mm	2	PED02	FUI85095	1
70mm	3	PED03	FUI85094	1
76mm	4	PED04	FUI85067	1
83mm	5	PED05	FUI85080	1




SHAATZ

Uterine Prolapse grade 2-3 - Cystocele



SIZES (MM)	SIZE CONVERSION	MEDICARE COLGATE CODE (MPC)	NHS PRODUCT CODE (NPC)	UNITS (UOI)
38mm	0	PESH00	FUI85091	1
45mm	1	PESH01	FUI85069	1
51mm	2	PESH02	FUI85071	1
57mm	3	PESH03	FUI85074	1
64mm	4	PESH04	FUI85097	1
70mm	5	PESH05	FUI85093	1
76mm	6	PESH06	FUI85073	1
83mm	7	PESH07	FUI85068	1
89mm	8	PESH08	FUI85092	1

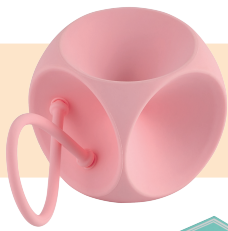


CUBE WITHOUT DRAIN

Uterine Prolapse grade 2-3 - Cystocele - Rectocele



SIZES (MM)	SIZE CONVERSION	MEDICARE COLGATE CODE (MPC)	NHS PRODUCT CODE (NPC)	UNITS (UOI)
25mm	0	PECU00	FUI85088	1
29mm	1	PECU01	FUI85085	1
33mm	2	PECU02	FUI85084	1
37mm	3	PECU03	FUI85083	1
41mm	4	PECU04	FUI85064	1
45mm	5	PECU05	FUI85066	1
50mm	6	PECU06	FUI85065	1
56mm	7	PECU07	FUI85087	1

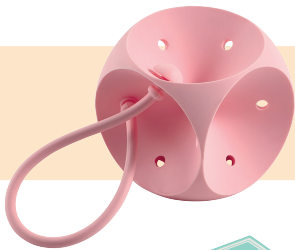

CUBE WITH DRAIN

Uterine Prolapse grade 2-3 - Cystocele - Rectocele



SIZES (MM)	SIZE CONVERSION	MEDICARE COLGATE CODE (MPC)	NHS PRODUCT CODE (NPC)	UNITS (UOI)
25mm	0	PECUD00	FUI85086	1
29mm	1	PECUD01	FUI85082	1
33mm	2	PECUD02	FUI85075	1
37mm	3	PECUD03	FUI85099	1
41mm	4	PECUD04	FUI85072	1
45mm	5	PECUD05	FUI85090	1
50mm	6	PECUD06	FUI85089	1
56mm	7	PECUD07	FUI85070	1

MediCare Colgate Pessary sizes

MediCare Colgate pessaries should be prescribed and fitted by a medical professional.

At Medicare Colgate we understand the challenges associated with identifying the optimal pessary fit, recognising that this process often involves a degree of trial and error. While our extensive pessary range is designed to accommodate a spectrum of pelvic organ prolapse grades, we have developed a guide to assist medical professionals in the selection of the most suitable pessary type. We have included alignment to other manufacturers' sizes to support decision-making. See the size conversation column on product descriptions.

The pessary is the right fit if:

- ✔ It does not feel uncomfortable.
- ✔ The patient can bear down, and it does not fall out.
- ✔ The patient has no difficulty going to the bathroom.

Contraindications for the use of Medicare Colgate Pessary

The pessary should not be inserted if the patient presents with acute genital tract infections, pelvic infections, or if the patient is non-compliant.

Cleaning Medicare Colgate Pessaries

Before insertion of a pessary, it is important to clean thoroughly the outer surface. It is recommended to wash the pessary with non-allergenic, antibacterial soap & rinse with water, and dry before initial use. There is no requirement to sterilise before use. Medicare Colgate pessaries are not suitable to be autoclaved and are designed for single-patient use.

Fitting procedure

All Medicare Colgate pessaries are a Class IIa medical device.

Fitting requires a trial of various sizes to determine the correct pessary size. A vaginal lubricant may be required for the patient's comfort before insertion of the pessary. The medical professional should determine if the patient requires oestrogen therapy before prescribing a pessary.

1. A medical professional should perform a normal pelvic examination before the fitting of a pessary. A pelvic examination helps determine the appropriate size that is required. The pessary will easily fold across the largest drainage holes to help with insertion and removal. The pessary should be large enough for its indication, but not cause unnecessary pressure or discomfort.
2. A medical professional should lubricate only the entering end of the pessary, compress, and gently insert through the introitus. Once the pessary has passed the introitus, release the compressed pessary so that it expands to its normal shape.
3. A medical professional should gently move the pessary to the cervix to support the cystocele or rectocele and the uterus. Once in the correct position, the physician should be able to insert an examining finger between the outer edge of the pessary and the vaginal wall. This spacing will ensure the patient's comfort and reduce the risk of pressure necrosis.
4. Once in place, the pessary should not dislodge when standing, sitting, squatting, or bearing down. It should not be uncomfortable for the patient during any of these routine activities.
5. If the patient is to care for the pessary, please inform on its cleaning, insertion, and removal.
6. It is important to inform the patient to report any discomfort or difficulty in urinating or defecating immediately.

Removal, cleaning & suggested follow up:

1. To remove the pessary, carefully move the pessary forward towards the introitus. When the pessary is within reach, compress the pessary and carefully withdraw.
2. The pessary can be cleaned with mild soap and warm water. The usage life of the pessary is limited. Examine pessary for signs of deterioration.
3. Discuss with the patient the importance of following instructions and the expected length of time for pessary usage.
4. Within 24 to 48 hours, ensure the patient is not allergic to the pessary. Examine the vagina and ask the patient if there has been any discomfort, irritation, pressure, sensitivity, or unusual vaginal discharge. Also, determine if there has been any improvement in personal symptoms.



MediCare
Colgate



Supplied by:

t +44 (0)1884 266 666

e info@sterifeed.com

medicarecolgate.com

facebook.com/sterifeed 

linkedin.com/company/medicare-colgate-sterifeed 

Instagram.com/medicare_colgate_gynae 

Medicare Colgate accepts no liability for loss or damage and will not replace or reimburse for damages when they are caused by using this product in any other manner.

Keep all components out of sight and reach of children.

MediCare Colgate LTD.
Post Cross Business Park, Kentisbeare, Cullompton,
Devon. EX15 2BB. United Kingdom